


Be in control of implant removal.
OPERACE. Flexible and efficient.

Mastering Implant Removal

Handling Technique



OPERACE
by PB Swiss Tools

 100% Made in Switzerland

PB SWISS TOOLS

Distributed by:

 **DePuy Synthes**
PART OF THE *Johnson & Johnson* FAMILY OF COMPANIES

Applications

Screw Recess Trial

Sterile single-use trials. To determine the correct size of the sterile screwdriver insert.

Screw head Hex


Screw	1.3mm, 1.5mm, 1.8mm, 2.0mm, 2.5mm,
recess size	3.0mm, 3.5mm, 4.0mm, 3/16", 5.0mm

Screw head Torx®/StarDrive®


Screw	T4, T5, T6, T7, T8, T9, T10, T15, T20, T25,
recess size	T25, T30

Ordering Information

Sterile single-use trials

-  80295 Screw recess trial for hexagonal socket screws, Hex 1.3–5.0



-  80296 Screw recess trial for screws Torx® and StarDrive®, T4–T30




Handling

Estimate the possible trial size from the bundle in accordance with the screw recess size and snap it out.




Check the size by inserting the trial and turning it slightly. If too small, choose a larger one.


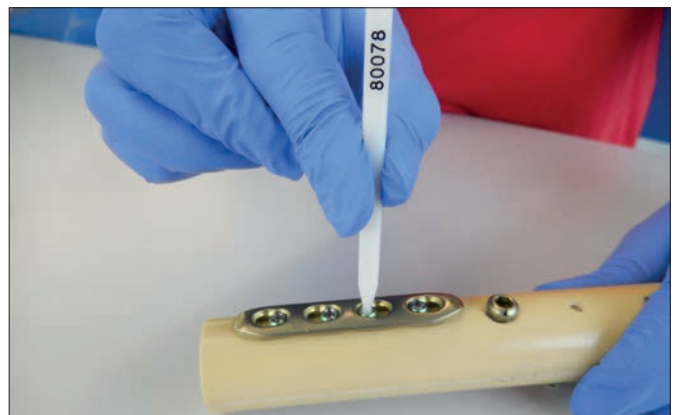


If the trial is too large, choose a smaller one from the bundle.

The trial must fit in the screw recess properly.

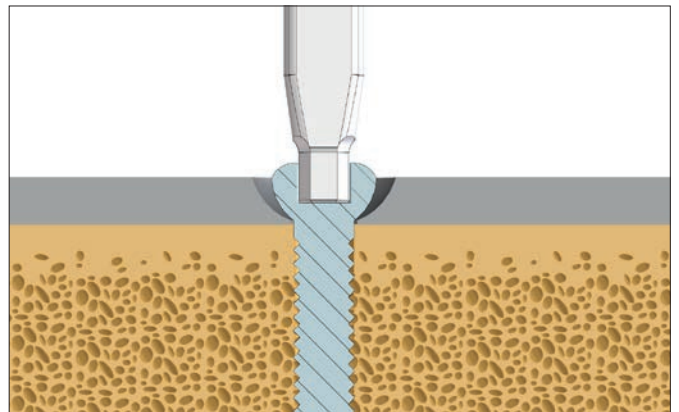

Determine the correct sterile screwdriver insert using the corresponding reference.

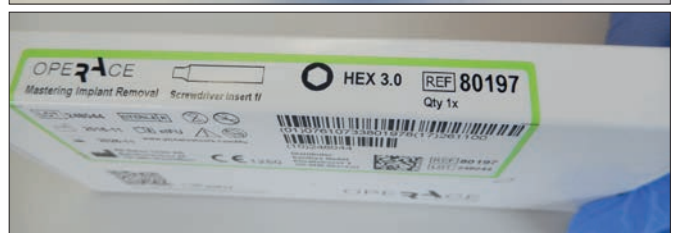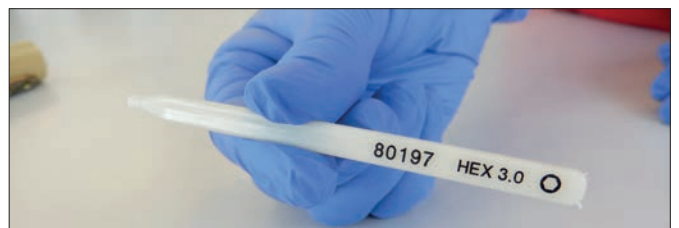

PB SWISS TOOLS



Legal Manufacturer:
PB Swiss Tools AG
Bahnhofstrasse 24
CH-3457 Wasen
info@pbswisstools.com



Information



Distributed by:
Synthes GmbH
Eimattstrasse 3
4436 Oberdorf
Switzerland
Tel: +41 61 965 61 11
Fax: +41 61 965 66 00
www.depuysynthes.com

Not all products are currently available in all markets.

This publication is not intended for distribution in the USA.

All surgical techniques are available as PDF files at
at www.pbswisstools.com/ifu

