

Be in control of implant removal.  
OPERACE. Flexible and efficient.

# Mastering Implant Removal

Handling Technique



**OPERACE**  
by PB Swiss Tools



100% Made in Switzerland

**PB SWISS TOOLS**

Distributed by:



**DePuy Synthes**

PART OF THE *Johnson & Johnson* FAMILY OF COMPANIES

## Screw Recess Trial

Sterile single-use trials. To determine the correct size of the sterile screwdriver insert.

### Screw head Hex



Screw	1.3 mm, 1.5 mm, 1.8 mm, 2.0 mm, 2.5 mm,
recess size	3.0 mm, 3.5 mm, 4.0 mm, 3/16", 5.0 mm

### Screw head Torx®/StarDrive®

Screw	T4, T5, T6, T7, T8, T9, T10, T15, T20, T25,
recess size	T25, T30

### Ordering Information

#### Sterile single-use trials

-  80295 Screw recess trial for hexagonal socket screws, Hex 1.3–5.0
-  80296 Screw recess trial for screws Torx® and StarDrive®, T4–T30




# Handling

Estimate the possible trial size from the bundle in accordance with the screw recess size and snap it out.




Check the size by inserting the trial and turning it slightly. If too small, choose a larger one.


If the trial is too large, choose a smaller one from the bundle.

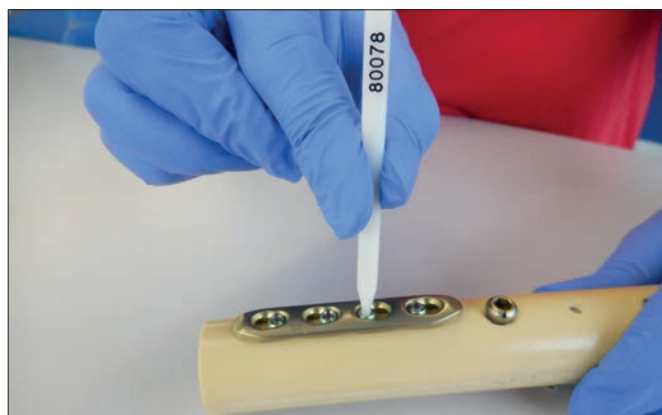

The trial must fit in the screw recess properly.

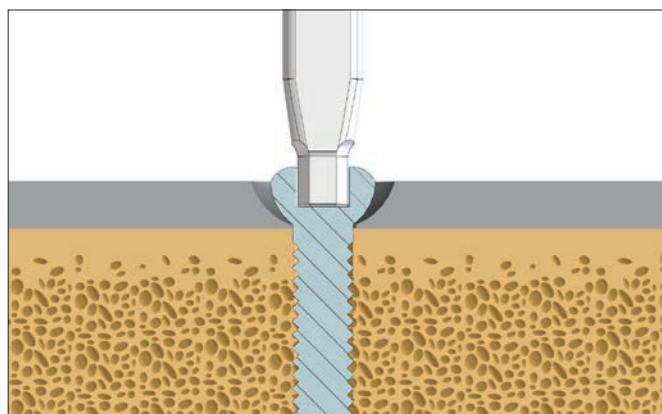

Determine the correct sterile screwdriver insert using the corresponding reference.

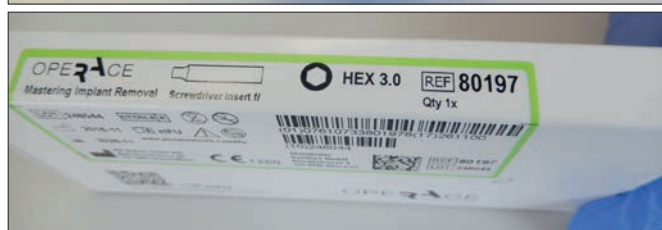

**PB SWISS TOOLS**



Legal Manufacturer:  
PB Swiss Tools AG  
Bahnhofstrasse 24  
CH-3457 Wasen  
[info@pbswisstools.com](mailto:info@pbswisstools.com)

EC REP

MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany



Information